



AreaCare Pressure Surface ONE-PIECE COVER SET-UP

Pressure surface is made from premium multi-density, high resilience memory foam material, combined with high quality vapour permeable, water resistant polyurethane cover

Minor assembly required: product is vacuum packed for economical shipping and compact storage.



- Please read these instructions before use
- Ensure product is dry prior to use
- Do not fold or roll the foam after reconstitution
- Do not allow sharp objects to penetrate the cover
- DO NOT** open the zippered mattress top cover during cleaning, infection control procedures or during use

Pertinent to

- BSPS3303
- BSPS3302
- BSPS6001-S
- BSPS6001-KS

Indications

A pressure surface for general use in:

- Home healthcare
- Professional healthcare

Intended for the prevention and treatment of skin breakdown and pressure injuries in patients of low risk.

Contraindications

- Unstable spinal cord injury
- Cervical traction

Working conditions

- Temperature: 10°C - 40°C
- Humidity: 30% - 75%

Shipping/Storage conditions

- Temperature: 10°C - 60°C
- Humidity: 10% - 80%

Warranty

Novis Healthcare Pty Ltd warrants the body (foam) of this product for a period of 6 years, and the cover for a period of 2 years.

Please refer to our Warranty Statement for full details.

Box Contents and Inspection

Carefully open box, avoid using a sharp blade which may damage the contents. Remove all components from the box - do not remove foam from the vacuum packaging at this stage.

Remove the top cover from the plastic wrapping, taking care not to cut or damage the material. Inspect the cover to ensure the zipper is fully functional.



- Product must be removed from vacuum packaging prior to the 'Reconstitute By' date printed on the carton.
- Failure to do so will void the product warranty.
- Ensure that the work area is clean and free of any potential hazards.
- Place a clean sheet over the work area if unable to clean.
- Please dispose of all packaging in a responsible manner
- Therapeutic devices should only be used in accordance with manufacturers instructions and under the consent, supervision and management of a suitably qualified health professional.

Pressure care and patient handling specialists

novis.com.au / 1300 738 885



AreaCare Pressure Surface ONE-PIECE COVER SET-UP

Assembly


- 1 Take the vacuum packed mattress and place on a flat, clean surface.
- 2 Locate the mattress end and gently tear the outer vacuum sealed plastic to allow the mattress to unravel. Take care not to cut or damage the inner plastic sheath or foam.



- 3 Unroll foam surface and lay flat on work area.



- 4 Remove the inner plastic sheath. The foam material will begin to reconstitute by drawing air into its cavities.

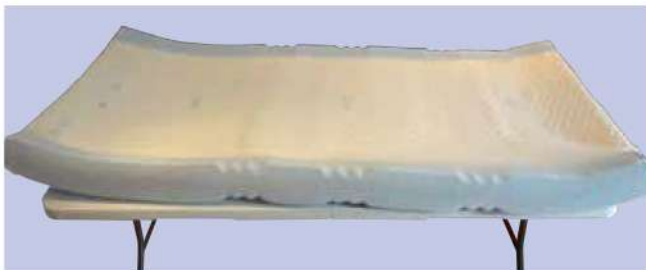

- 5 Fit cover without delay with "THIS SIDE UP" facing upwards, feed the cover onto the foam.



- 6 To feed, gently pull one side of the cover down about 10cm, and pull other side to match. Repeat this process until the cover is fully fitted.

Avoid putting undue stress on the seams or where the zipper is stitched to the cover.



- 7 Check the seams are located approximately half way between the top and bottom; gently realign cover if not.

Close zipper at foot end of cover and ensure waterfall flap (over zipper) is correctly positioned.




- 8 When finished, place the pressure surface on top of the bed platform, oriented as per the indicator label on top cover.



AreaCare Pressure Surface TWO-PIECE COVER SET-UP

Pressure surface is made from premium multi-density, high resilience memory foam material, combined with high quality vapour permeable, water resistant polyurethane cover

Minor assembly required: product is vacuum packed for economical shipping and compact storage.

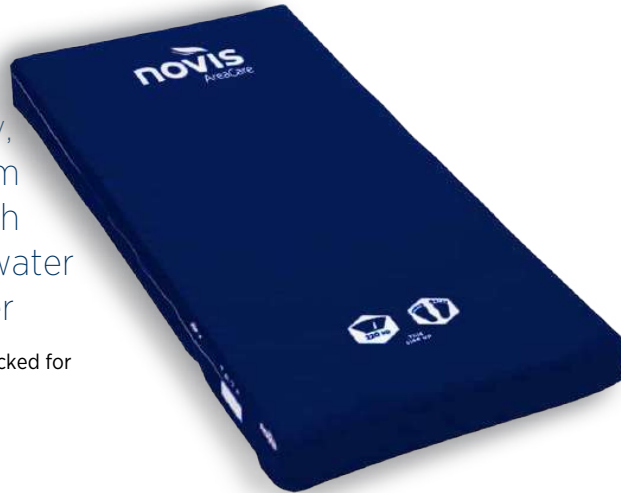

- ⚠ Please read these instructions before use
- ⚠ Ensure product is dry prior to use
- ⚠ Do not fold or roll the foam after reconstitution
- ⚠ Do not allow sharp objects to penetrate the cover
- ⚠ **DO NOT** open the zippered mattress top cover during cleaning, infection control procedures or during use

Pertinent to

- BSPS3300
- BSPS3000
- BSPS3001
- BSPS6003-S
- BSPS6003-KS
- BSPS6002-S
- BSPS6002-KS

Intended Use

Indications

A pressure surface for general use in

- Home healthcare
- Professional healthcare

Intended for the prevention and treatment of skin breakdown and pressure injuries in patients of low risk.

Contraindications

- Unstable spinal cord injury
- Cervical traction

Working conditions

- Temperature: 10°C - 40°C
- Humidity: 30°C - 75°C

Shipping/Storage conditions

- Temperature: 10°C - 60°C

Warranty

Novis Healthcare Pty Ltd warrants the body (foam) of this product for a period of 6 years, and the cover for a period of 2 years.

Please refer to our Warranty Statement for full details.

Box Contents and Inspection

Carefully open box, avoid using a sharp blade which may damage the contents. Remove all components from the box – do not remove foam from the vacuum packaging at this stage.

Remove the top cover from the plastic wrapping, taking care not to cut or damage the material. Inspect the cover to ensure the zipper is fully functional.



- ⚠ Product must be removed from vacuum packaging prior to the 'Reconstitute By' date printed on the carton.
- ⚠ Failure to do so will void the product warranty.
- ⚠ Ensure that the work area is clean and free of any potential hazards.
- ⚠ Place a clean sheet over the work area if unable to clean.
- ⚠ Please dispose of all packaging in a responsible manner
- ⚠ Follow manufacturers instructions and under the consent, supervision and management of a suitably qualified health professional.

Pressure care and patient handling specialists

novis.com.au / 1300 738 885



AreaCare Pressure Surface TWO-PIECE COVER SET-UP

Assembly

- 1 Unwrap and separate the top and bottom covers. Lay the bottom cover on a flat level surface, ensuring handles are positioned on the outside (underneath) and the zipper 'opening' ('start/stop') is at the foot end.



- 2 Place the vacuum packed mattress on top of the bottom cover.

- 3 Locate the mattress end and gently tear the outer vacuum sealed plastic to allow the mattress to unravel. Take care not to cut or damage the inner plastic sheath or foam.



- 4 Unroll foam surface and lay flat on the bottom cover.




- 5 Remove the inner plastic sheath. The foam material will begin to reconstitute by drawing air into its cavities.




- 6 Ensure "THIS SIDE UP" is facing upward, and "HEAD END" is at the head end if applicable. The foot end of the cover is where the zipper starts and ends - therefore the Head end of the foam should be aligned with the end of the cover without the zipper.



- 7 Place the top cover over the mattress, ensuring the zipper orientation is maintained.

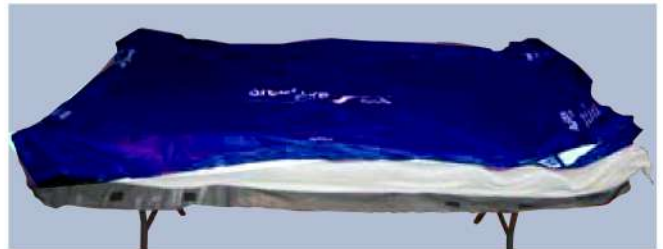

Zip the top and bottom covers together. Avoid putting undue stress on the seams or where the zipper is stitched to the cover.

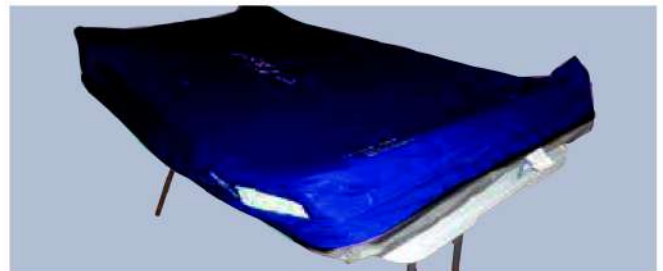

- 8 Check the seams are located approximately half way between the top and bottom; gently realign cover if not.

- 9 Ensure the waterfall flap is folded down on all four sides - the base should be fully hidden




- 10 When finished, place the pressure surface on top of the bed platform, oriented as per the printed label on top cover



AreaCare Pressure Surface CARE AND CLEANING

Pressure surface is made from premium foam material, combined with high quality vapour permeable, water resistant polyurethane cover

Minor assembly required: product is vacuum packed for economical shipping and compact storage.

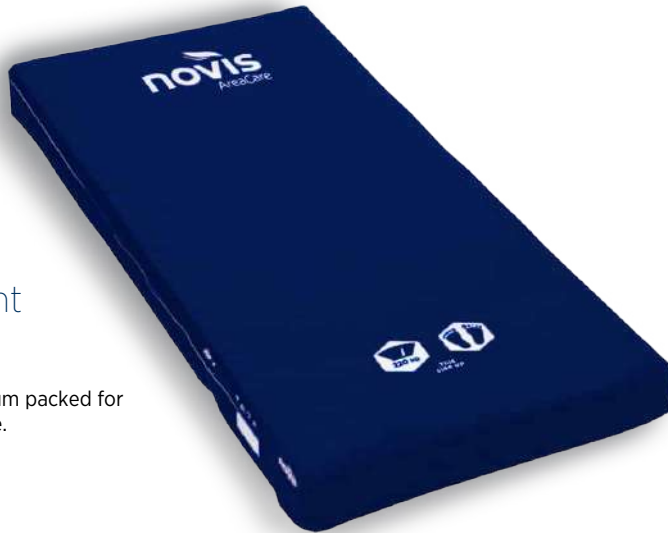

- ⚠ Please read these instructions before use
- ⚠ Ensure product is dry prior to use
- ⚠ Do not fold or roll the foam after reconstitution
- ⚠ Do not allow sharp objects to penetrate the cover
- ⚠ **DO NOT** open the zippered mattress top cover during cleaning, infection control procedures or during use

Regular Maintenance

Regularly check the surface for damage and wear, paying special attention to the seams and zipper. Do not use if the top cover is torn, cracked or deformed in any way.

Ensure foam is not deformed or compressed in any way.

- ⚠ Top cover cannot be repaired if damaged

Care & Cleaning

Laundry powder residue, starch and skin cells can act as an abrasive, therefore we recommend the cover be cleaned weekly.

- | | |
|---|---|
| ♻ To clean cover, wipe clean with a mild soap and warm water solution and rinse well. Use of a Sanitising/Deodorising Mattress Cleaning solution such as Sterri-Matt® is also acceptable. | ⊗ Do not use bleach, harsh abrasives or phenol based cleaning products. |
| ☒ Dry with a clean cloth. Ensure cover is dry before use. | ⊗ Do not iron. |
| ⊗ Do not dry clean. | ⊗ Do not wring out or twist. |
| ⊗ Do not tumble dry. | 🧺 Top cover is machine washable up to 80°C. |
| | ⊗ Bottom Cover (if applicable) is not machine washable. |
| | ⊗ Foam is not washable. |

In the unlikely event that the cover has been breached (and foam has been soiled) product replacement should be considered.

- ⚠ Cleaning with the patient on the mattress is not recommended
- ⚠ Ensure product is thoroughly cleaned between patients to reduce the potential of cross-contamination

Storage

Prior to storage, ensure the product is clean and dry. Store in a clean, dry, dust free area.

Do not place heavy objects on surface when not in use. We recommend covering the product with a soft, clean cloth or other protective surface.

Warranty

Novis Healthcare Pty Ltd warrants the body (foam) of this product for a period of 6 years, and the cover for a period of 2 years.

Please refer to our Warranty Statement for full details.

Product must be removed from vacuum packaging prior to the 'Reconstitute By' date printed on the carton.

Failure to do so will void the product warranty.