



STARlock Cushion Setup Guide

	Open valve above red bulb on STARlock
	Press the button on the bulb for 5 seconds.
5 500	Sit on the cushion and press again for 5 seconds to ensure all air is removed from the locking system.
	Open the silver valve on the cushion corner by turning it anticlockwise
	Inflate STAR cushion full and leave to rest for 10 minutes with the valve open
ILL C	Close the silver valve by turning clockwise
2.3 cm 3/4-1"	Position the person on the cushion and conduct a bottoming-out test to ensure there is 2-3cm of air below the Ischial Tuberosities
	If cushion is overinflated, turn the valve anticlockwise to release air until 2-3 cm of air remains under Ischial Tuberosities
	When correct height is achieved ensure silver valve is closed by turning it clockwise





