Etac Cloo

Manual

etac®


78224.9 23-01-16

sv no da en de fi fr nl it es


A


8030 1204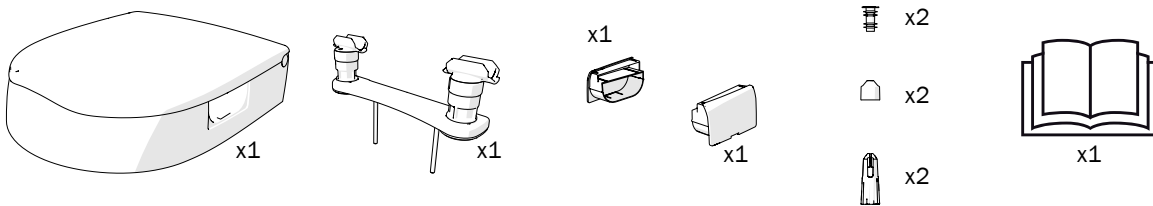


8030 1214

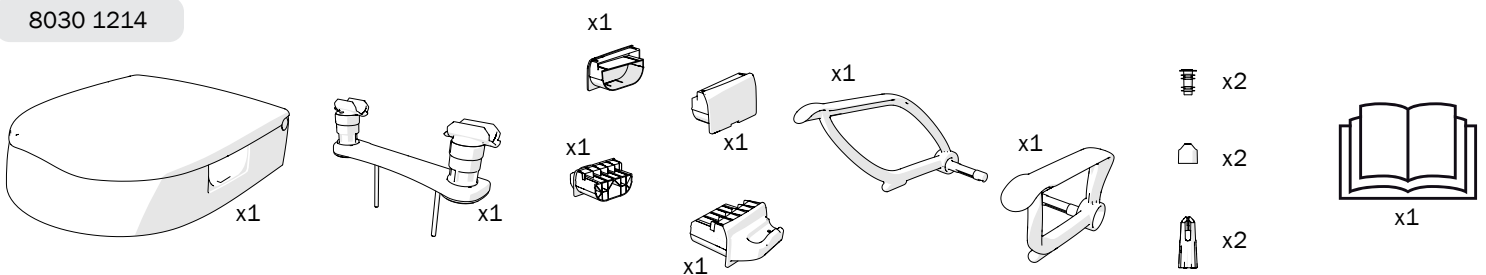
B






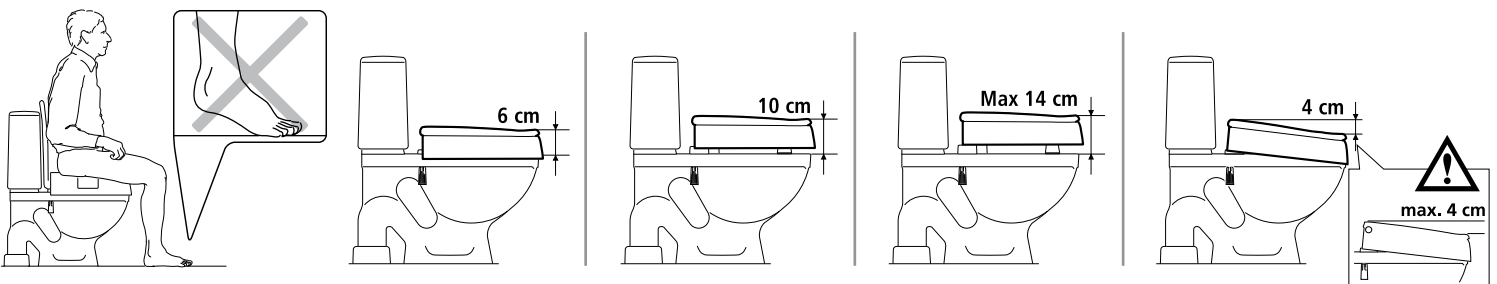
C





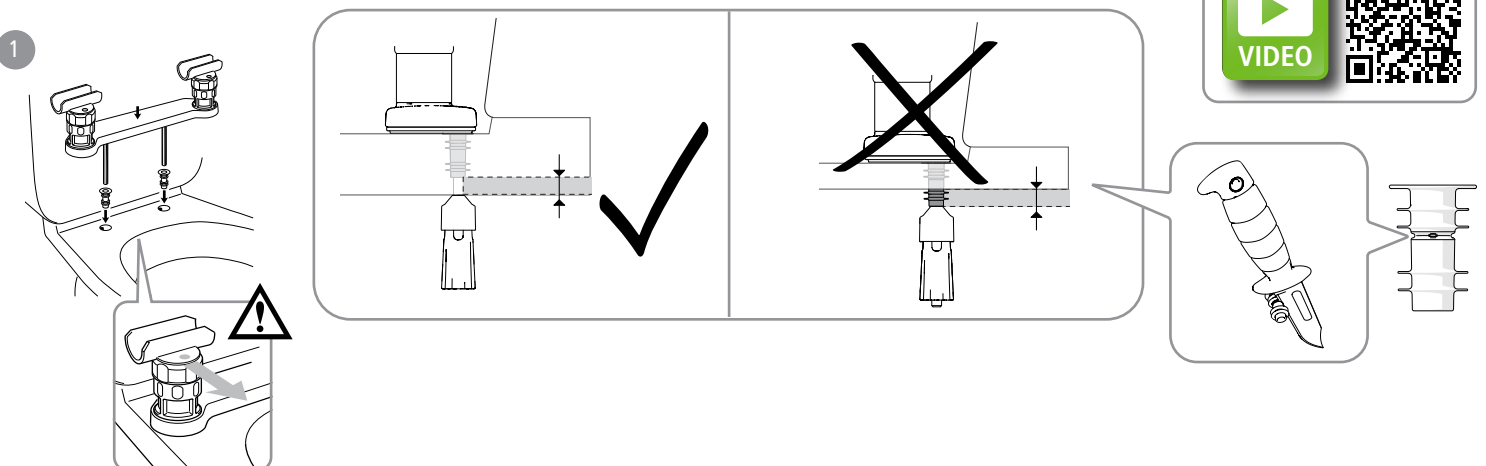
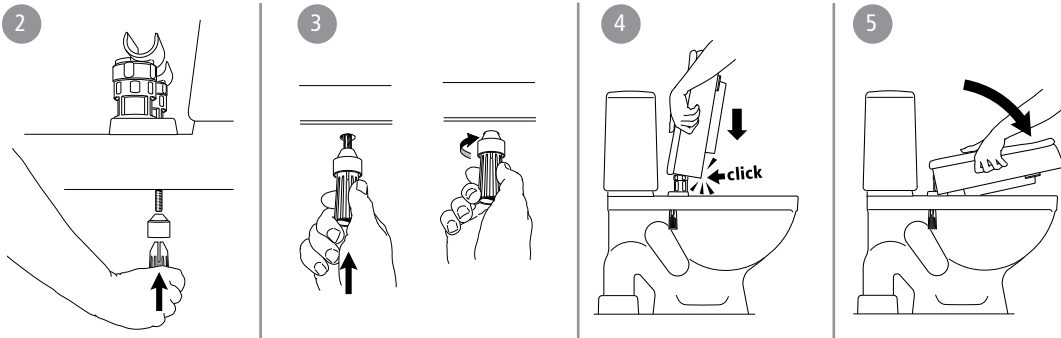
D


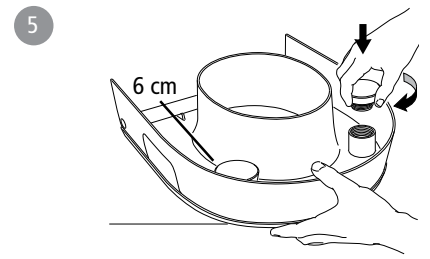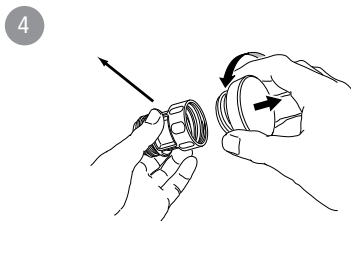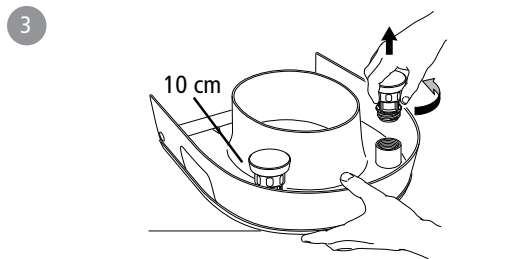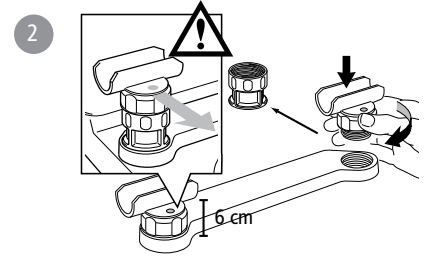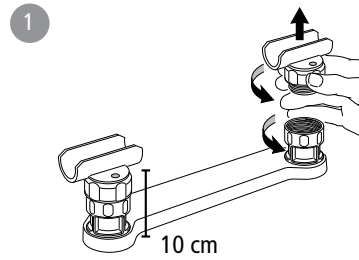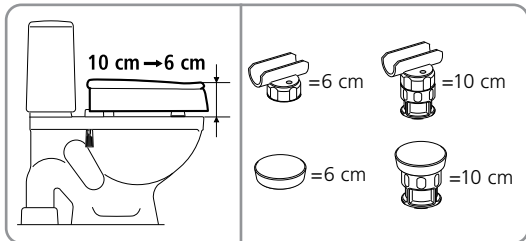
E

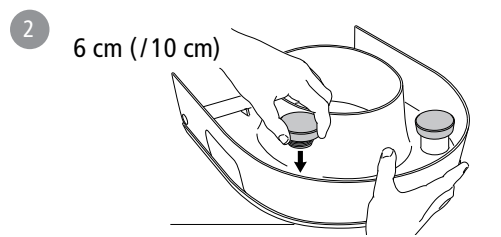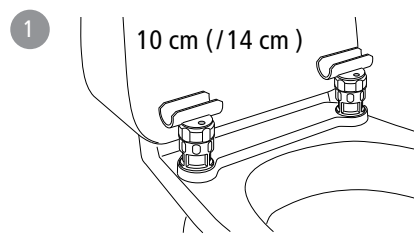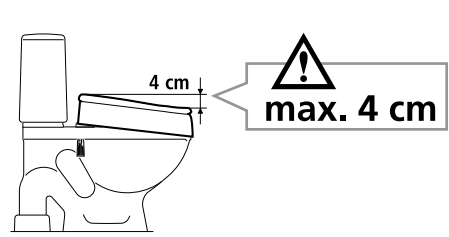
F

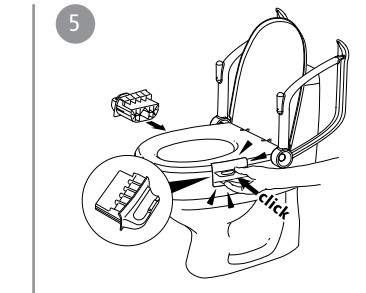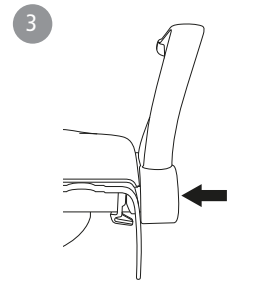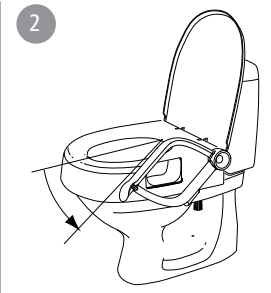
G

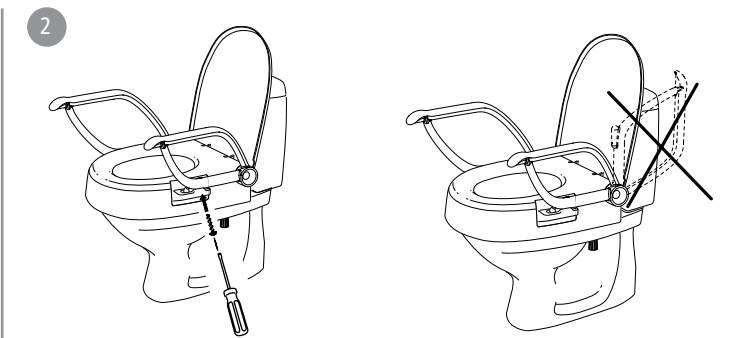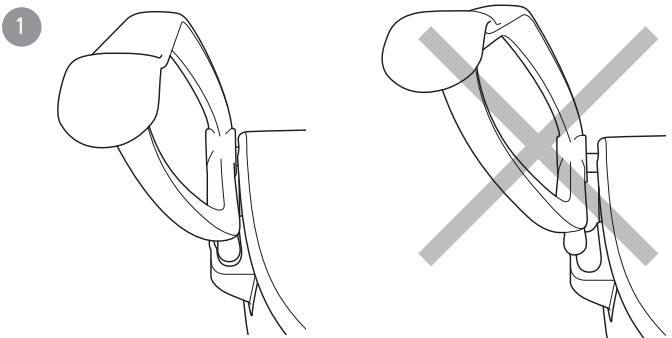
H

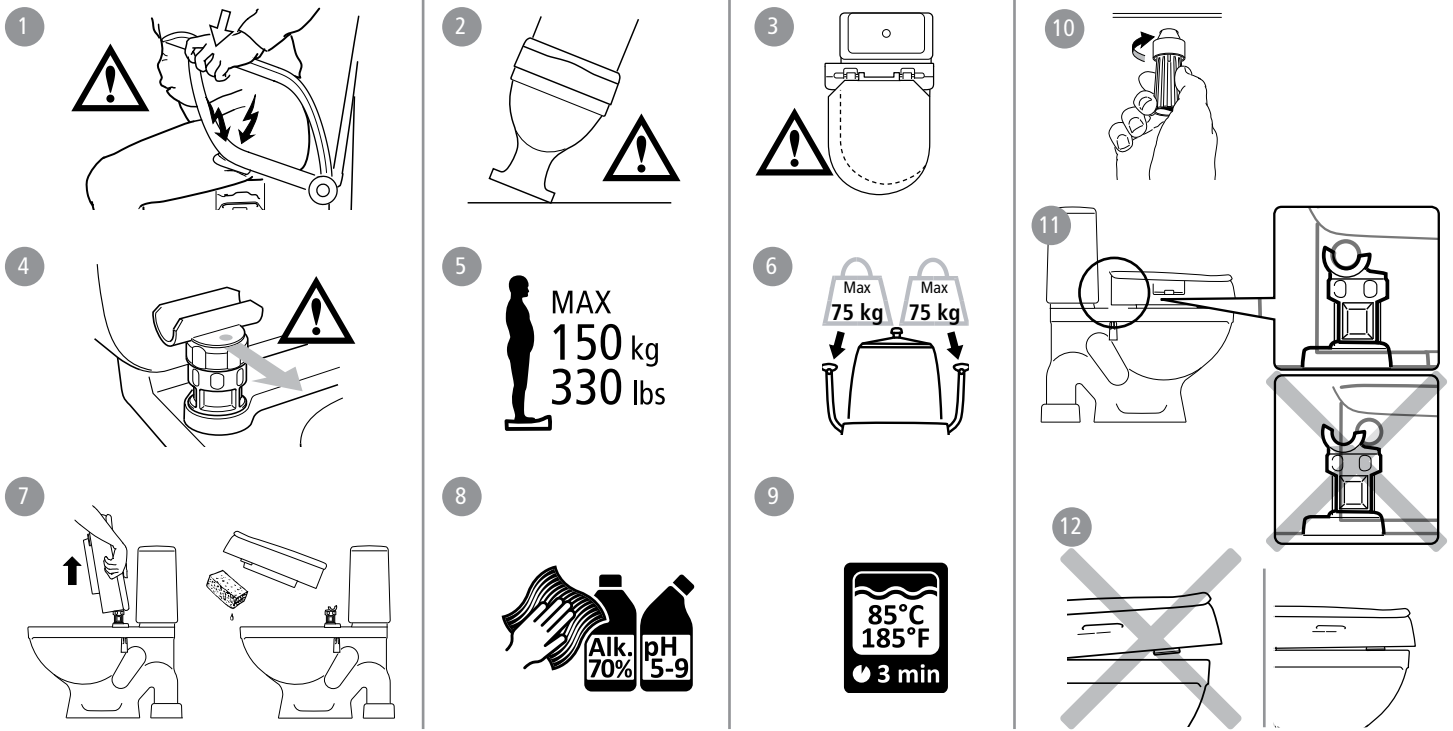
I

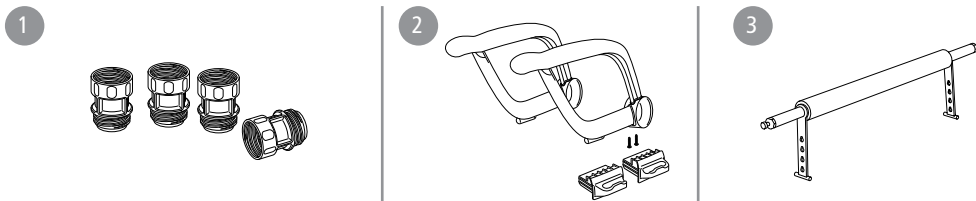
J

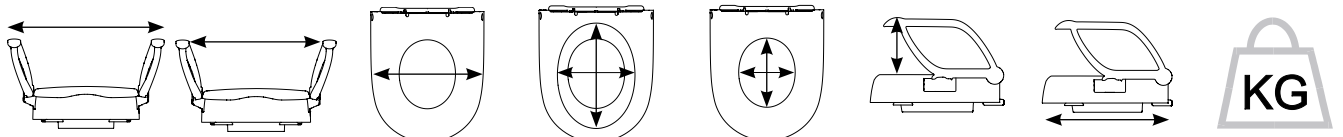80301204	-	-	400 mm	220/290 mm	195/240 mm	-	-	3,3 kg
80301214	590 mm	490 mm	400 mm	220/290 mm	195/240 mm	205 mm	460 mm	4,8 kg

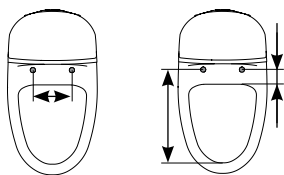80301204	Max 185 mm	Min 80 mm Max 345 mm						
80301214	Max 185 mm	Min 80 mm Max 345 mm						

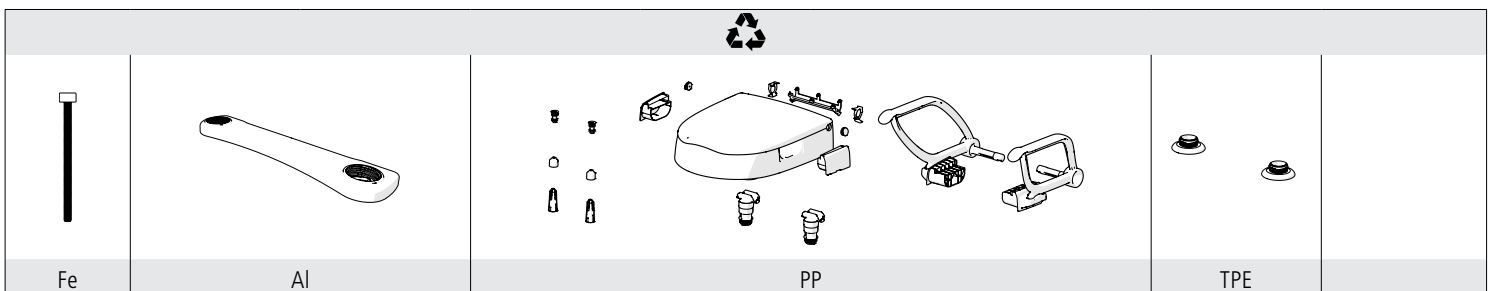Svenska

Allmänt

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en English

General information

Thank you for choosing an Etac product. In order to avoid damage during assembly, handling and use, it is important to read this manual and save it for future reference. You can also find it at www.etac.com. You can select your language via the "International" and "Local websites" link. Here you will also find other product documentation, such as prescriber information, pre purchase guide, and reconditioning instructions.

Intended Use

Cloo is a height-adjustable toilet seat raiser with fixed mounting, which can be used with one or two arm supports, as well as without arm support. The arm supports are foldable, but can also be fixed in the unfolded position. Cloo can also be mounted at an angle.

The arm supports provide increased security while sitting, and support while sitting down/standing up from the toilet. This product is intended for users with limited ability to sit down and/or stand up from the toilet due to reduced mobility or strength.

The product is intended for adult users, generally persons with a height of at least 146 cm and a weight of at least 40 kg.

The product is suitable for standard toilet seats. For exact measurements, see the table under figure J, "Technical data".

This product is intended for use in domestic environments and institutions, and may not be used in any way other than that stated above. There are no known contraindications. The product meets the requirements of the Medical Device Regulation (EU) 2017/745.

In case of an adverse event occurred in relation to use of the device, it should be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to the manufacturer.

Warranty 5 year warranty against material and manufacturing defects. For terms and conditions, see www.etac.com. A defective product must not be used.

Storage

The product should be stored indoors in a dry place at a temperature above 5°C. If the product has been stored for a long time (more than four months), its function must be checked by an expert before use.

Service life 10 years. For complete information regarding the service life of the product, see www.etac.com.

Content of the delivery [Figure A](#)

Choosing your height [Figure B](#)

Mounting..... [Figure C](#)

Height adjustment..... [Figure D](#)

Angled fitting..... [Figure E](#)

Armrest [Figure F](#)

Fixed mounting..... [Figure G](#)

Safety and Maintenance..... [Figure H](#)

H1. Risk of crushing **H2.** The commode must be securely anchored **H3.** The product should be mounted centrally on the commode **H4.** Mount so that the red dot is pointing forwards. **H5.** Maximum user weight 150 kg **H6.** The armrests have each been tested and approved for half of the maximum user weight **H7.** The seat ring can be removed for cleaning. Make sure it clicks in place completely when refitting **H8.** Clean the product with a solvent-free cleaning agent with a pH level between 5 and

9, or with a 70% disinfectant solution. **H9.** Can be decontaminated at max. 85°C if so required. **H10.** Check the screw attachment.

In case of an adverse event occurred in relation to use of the device, it should be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to the manufacturer.

Accessories..... [Figure I](#)

11. Blocks for 14 cm elevation. **12.** Armrest kit. Supplied in pairs. **13.** Crossbar

Technical data..... [Figure J](#)

The product's date of manufacture can be read from the barcode on the product. The number 11 is shown below the barcode in brackets. The number combination after these brackets is the date of manufacture.

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Etac Supply Center AB